

A REVOLUTIONARY 3-WIRE SEQUENCE

SAGITTAL FIRST

Integral to the **SAGITTAL FIRST™** Treatment Philosophy is the use of the **Carriere Motion 3D™** Appliance to correct AP discrepancies and stimulate initial tooth movement.

This fosters easier transitions into larger wires earlier in treatment, less time in braces, and shorter overall treatment times.

M-SERIES WIRES

Introducing the All-New **Carriere M-Series™ Wires**, including .015 **Cu™ Nitanium® Wire** and the innovative 3-Wire Sequence. **M-Series Wires** are designed to:

- Enable a simplified wire progression protocol
- Reduce the number of wires used
- Finish cases in about 12 months

MOTION 3D APPLIANCE

3–6 MONTHS

CLASS I PLATFORM ESTABLISHED

M-SERIES WIRES

8–10 MONTHS



METAL BRACKET SYSTEMS – M-SERIES WIRE SEQUENCE

WIRE NAME	DESCRIPTION	OBJECTIVE	TIME IN TREATMENT	NUMBER OF APPTS. (5-6 WEEK INTERVALS)	ITEM NUMBER
1 M-ONE	.015 Cu Nitanium 27° with EZ Stops™ Delivery System	Begin leveling, aligning, and rotations	5-6 weeks	1 appointment	103-M1
2 M-TWO	.020 X .020 Cu Nitanium 35° with EZ Stops Delivery System	Complete leveling, aligning, and elimination of rotations. Arch development is initiated, 3 dimensional control of wire will begin initial torque control	6-18 weeks, based on complexity and clinical judgement	2-3 appointments	103-M2
3 M-THREE	.019 X .025 Beta Titanium	Detailing and finishing	8-12 weeks	2-3 appointments	103-M3B

NEW SIZE

CERAMIC BRACKET SYSTEMS – M-SERIES WIRE SEQUENCE

Follow the 3-Wire Sequence and add the wire below prior to using the M-THREE.

WIRE NAME	DESCRIPTION	OBJECTIVE	TIME IN TREATMENT	NUMBER OF APPTS. (5-6 WEEK INTERVALS)	ITEM NUMBER
M-TWO.FIVE	.019 X .025 Cu Nitanium 35° with EZ Stops Delivery System	For final bracket alignment prior to M-THREE BETA wire with ceramic brackets	3-4 weeks	1 appointment	103-M2.5

OPTIONAL WIRES

WIRE NAME	DESCRIPTION	OBJECTIVE	TIME IN TREATMENT	NUMBER OF APPTS. (5-6 WEEK INTERVALS)	ITEM NUMBER
M-ONE.FIVE	.018 Cu Nitanium 27° with EZ Stops Delivery System	Used in a small number of cases for stubborn rotations prior to placing M-TWO wire	4-6 weeks	1-2 appointments	103-M.5
M-THREE STEEL	.019 X .025 Stainless Steel	Surgical stabilization wire placed prior to surgery and/or to complete expansion in stubborn cases; used instead of M-THREE BETA	8-12 weeks	2-3 appointments	103-M3S

THE SAGITTAL FIRST REVOLUTION

Simplify & shorten treatment with predictable results.

The **SAGITTAL FIRST Philosophy** is a ground-breaking approach that simplifies and shortens Class II and Class III treatments. It employs the **MOTION 3D Appliance** to treat the AP dimension at the beginning of treatment before placing brackets or aligners. By resolving the most difficult part of treatment first, you can achieve a Class I platform in 3 to 6 months, shortening total treatment time by a minimum of 6 months. You know how excited patients and parents become when you mention shorter treatment times!

PREDICTABLE

MOTION 3D Class II Appliance

INNOVATIVE

SLX 3D Bracket System

ATTACH-LESS

SLX Clear Aligners



CARRIERE® | SLX® 3D



CARRIERE® | SLX® 3D



THE PERFORMANCE REVOLUTION

Designed for you,
your staff & your patients
with no compromises.





1

DO THE TWIST

With a feather touch and slight twist of the instrument, the slide is propelled to the open and locked position.

2

COVER IT UP

Full M/D slot coverage provides unmatched rotational control, to help ensure that you finish faster.

3

FEEL THIS

The smoothest brackets we've ever made. Incredibly comfortable.

UNMATCHED CLINICAL CONTROL

The Complete Carriere SLX 3D Self-Ligating Bracket System

The dedication to practice efficiencies and to advancing the state-of-the-art patient care will be evident when you experience the beauty, comfort, and extraordinary performance of the **Carriere® SLX® 3D System**.



■ Don't bend wire, like a pro

With proven and precise torque values, in/out thicknesses and slot depths, the **SLX 3D Bracket** is designed to ensure less wire bending so you can spend more time doing anything but wire bending.



WIDE

Centrals & cuspids



MEDIUM

Bicuspids



NARROW

Lower & upper laterals

■ One size doesn't fit all

The **SLX 3D Bracket System** has narrow brackets for narrow teeth, and wide brackets for wide teeth - because size matters when it comes to rapidly securing rotational control and ideal tooth positioning.



CLEAR

Mushroom shaped pylons for secure retention



■ Guard rails aren't just for highways

Our exclusive Adhesive **Guard Rail™ Technology (AGR)** directs excess adhesive to the sides for fast, easy clean up.



METAL

Micro-etched pylons



■ Easy on, easy off

Our compound contour bases are designed to “fit like a glove,” while the beveled base edges aid in quick and clean removal.



EZ TWIST

To the open and locked position



■ Just do the twist

NEW EZ Twist Tool
NOW AVAILABLE

Our proprietary new **EZ Twist®** feature brings a new level of staff joy and patient comfort to your practice. For fast, comfortable wire changes, **JUST DO THE TWIST!**



SELF-LIGATING



Buccal Tubes



■ A complete self-ligating system

SLX 3D Self-Ligating Buccal Tubes incorporates all of the amazing features of **SLX 3D Brackets**, including HSO's first **clear ceramic upper first molars!**

EZ INVENTORY CONTROL

MAXILLARY							SLX 3D Metal 		SLX 3D Clear 	
	Description	Color-Code	Tooth#	Torque	Ang	Antirotation	.022 R	.022 L	.022 R	.022 L
CENTRALS	—	BLACK	U1	+17°	+5°	0°	777-UR1H-10	777-UL1H-10	776-UR1H-5	776-UL1H-5
	—	BLACK	U1	+12°	+5°	0°	777-UR1-10	777-UL1-10	776-UR1-5	776-UL1-5
LATERALS	—	PINK	U2	+12°	+8°	0°	777-UR2H-10	777-UL2H-10	776-UR2H-5	776-UL2H-5
	—	PINK	U2	+8°	+8°	0°	777-UR2-10	777-UL2-10	776-UR2-5	776-UL2-5
CUSPIDS	Hook	GREEN	U3	+7°	+9°	0°	777-UR3H-HK-10	777-UL3H-HK-10	776-UR3H-HK-5	776-UL3H-HK-5
	Hook	GREEN	U3	0°	+9°	0°	777-UR3-HK-10	777-UL3-HK-10	776-UR3-HK-5	776-UL3-HK-5
BICUSPIDS	—	PURPLE	U4 & 5's	-7°	0°	0°	777-U45-10	777-U45-10	776-U45-5	776-U45-5
	Hook	PURPLE	U4 & 5's	-7°	0°	0°	777-UR45-HK-10	777-UL45-HK-10	776-UR45-HK-5	776-UL45-HK-5
1 ST MOLARS	—	WHITE	U6's	-14°	0°	12°DO	977-UR6-HK-10	977-UL6-HK-10	976-UR6-5	976-UL6-5
2 ND MOLARS	—	GREY	U7's	-14°	0°	10°DO	977-UR7-HK-10	977-UL7-HK-10	—	—

MANDIBULAR							SLX 3D Metal		SLX 3D Clear	
	Description	Color-Code	Tooth#	Torque	Ang	Antirotation	.022 R	.022 L	.022 R	.022 L
ANTERIORES	—	YELLOW	L1&2's	-1°	0°	0°	777-L12-10	777-L12-10	776-L12-5	776-L12-5
	—	YELLOW	L1&2's	-6°	0°	0°	777-L12L-10	777-L12L-10	776-L12L-5	776-L12L-5
CUSPIDS	Hook	BLUE	L3	0°	+5°	0°	777-LR3-HK-10	777-LL3-HK-10	776-LR3-HK-5	776-LL3-HK-5
	Hook	BLUE	L3	-6°	+5°	0°	777-LR3L-HK-10	777-LL3L-HK-10	776-LR3L-HK-5	776-LL3L-HK-5
1 ST BICUSPIDS	—	GREY	L4	-12°	+2°	0°	777-LR4-10	777-LL4-10	776-LR4-5	776-LL4-5
	Hook	GREY	L4	-12°	+2°	0°	777-LR4-HK-10	777-LL4-HK-10	776-LR4-HK-5	776-LL4-HK-5
2 ND BICUSPIDS	—	RED	L5	-17°	+2°	0°	777-LR5-10	777-LL5-10	776-LR5-5	776-LL5-5
	Hook	RED	L5	-17°	+2°	0°	777-LR5-HK-10	777-LL5-HK-10	776-LR5-HK-5	776-LL5-HK-5
1 ST MOLARS	Hook	WHITE	L6's	-25°	0°	2°DO	977-LR6-HK-10	977-LL6-HK-10	—	—
2 ND MOLARS	Hook	YELLOW	L7's	-10°	0°	0°	977-LR7-HK-10	977-LL7-HK-10	—	—

CARRIERE KITS (Call for availability)				SLX 3D Metal		SLX 3D Clear	
Rx	Description	QTY of sets	Total brackets	Item numbers .022		Item numbers .022	
STD TORQUE	U/L 5×5 kit with hooks on 3's	1	20	777-KIT-HK		776-KIT-HK	
	U/L 5×5 kit with hooks on 3's, 4's, 5's	1	20	777-KIT-HKS		776-KIT-HKS	
	U/L 6×6 kit with hooks on 3's & 6's	1	24	777-KIT6-HK		776-KIT6-HK	
	U/L 6×6 kit with hooks on 3's, 4's, 5's, 6's	1	24	777-KIT6-HKS		776-KIT6-HKS	

ACCESSORIES	ITEM NUMBER
SLX 3D Typodont	631-777
SLX 3D Clear Typodont	631-776
SLX 3D EZ Twist Opener	201-510
PATIENT EDUCATION	
SLX 3D Patient Education Brochure	999-348



SLX EZ Twist Opener



SLX 3D Clear Typodont



SLX 3D Typodont